

INDICATIONS FOR USE:

The NeoMed Enteral Only Extension Set is intended for use as an extension set for nasogastric/oralgastric or gastric tube enteral feeding tubes, incorporating safety connectors which help mitigate the risk of accidental connection of an I.V. system to the enteral system or the enteral system to an I.V. system.

This device is indicated for use in neonatal and pediatric patients in connection with an enteral feeding tube to provide nutrition via nasal or oral gastric placements.

The Enteral Only Extension Set is a sterile disposable for single patient use only.






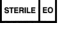




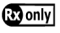
CONTRAINDICATIONS:

The NEOMED extension set is not intended for individual use longer than 24 hours. Replace and maintain administration set in accordance to hospital protocol and not to exceed NANN or ADA guidelines, as applicable.

DIRECTIONS:

1. Follow physician's instructions in opening package and use of contents. Use aseptic technique.
2. Verify gastrostomy tube compatibility prior to use.
3. Attach Enteral Only Connector (inlet) to outlet of administration set. Attach extension set outlet to enteral only feeding tube.
4. Close slide clamp and plug until ready to use or when not in use.

SYMBOLS:

-  Single Use Only. Do not reuse.
-  The device has no components which contain DEHP
-  The device has no components made of natural rubber latex
-  Do not use if package is damaged
-  Manufacturer
-  Sterilized using Ethylene Oxide
-  Catalog Number
-  Lot Number
-  Use By
-  Caution, consult accompanying documents
-  Prescription Only. Caution: Federal law restricts this device to sale by or on the order of a physician.

LBL-ENT-01 Rev4
Effective 08-2010