


NeoDrape® Fenestrated (Ref: ND-FS)  
Instructions For Use:

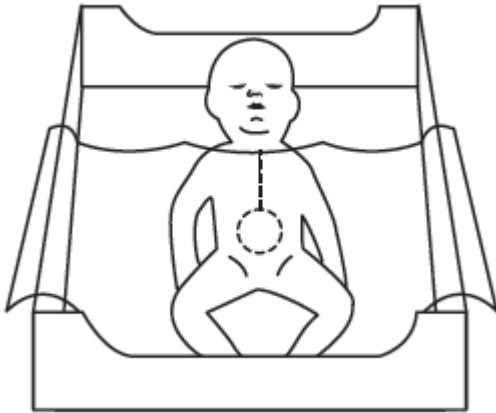

Follow hospital and accepted clinical practice use guidelines for proper use of a sterile drape.

LBL-NDFS-01 Rev. 2  
Effective: 07-2010

NeoDrape® Fenestrated is intended to be used for up to one hour after delivery or during procedures that have a duration of an hour or less.

Lay the NeoDrape® Fenestrated on top of the infant. Do not place the NeoDrape® Fenestrated over the infant's face, ensuring the infant's airway is clear at all times.

Align the opening/fenestration over the insertion site carefully with the long end of the drape set towards the infant's feet. The drape may also be placed underneath the patient to create a sterile barrier.

Remove the adhesive backing from both sides of the NeoDrape® Fenestrated and tape down to secure drape.

Following use, tear along the perforation from the fenestration toward the edge and remove drape.

To avoid overheating the infant, do not use the NeoDrape® Fenestrated with a warming mattress or chemical warming pad. Continue to monitor the infant's temperature during the use of the NeoDrape® Fenestrated.