

NeoDrape® (Ref: ND)

Instructions For Use:



Follow hospital and accepted clinical practice use guidelines for proper use and duration of an occlusive wrap.

Lay the infant on top of the NeoDrape®.

Fold one side of the NeoDrape® lightly over the infant and then fold other side on top. Do not place the NeoDrape® over the infant's face, ensuring the infant's airway is clear at all times.

Do not wrap the infant too tightly in the NeoDrape®.

Do not use the NeoDrape® for infants born with blistering skin conditions that preclude the use of an occlusive wrap.

To avoid overheating the infant, do not use the NeoDrape® with a warming mattress or chemical warming pad. Continue to monitor the infant's temperature during the use of the NeoDrape®.