NEOMED[®]

NeoBottle[®]

- EN Oral Dispenser - For Oral / Enteral Use - Instructions for Use
 FR Distributeur Oral - Pour un Usage Oral / Enteral - Mode d'emploi
 ES Dispensador Oral - Para uso Oral / Gástrico - Instrucciones de Uso
 PT Dispensador Oral - Para via Oral / Entérica - Instruções de Utilização
 DA Oral Beholder - Til Oral / Enteral Anvendelse - Brugsanvisning
 NL Orale Dispenser - Voor Oraal / Enteraal Gebruik - Gebruiksaanwijzing
 DE Oraldispenser - Für den Oralen / Enteralen Gebrauch – Gebrauchsanweisung
 NO Oral Dispenser - For Oral / Enteral Bruk - Bruksanvisning
 SV Flaska - För Bröstmjök för Oralt / Entertalt Bruk - Bruksanvisning

REF RÉF BEST.-NR.	Type/Tipo/Typ	Case Qty Qté boîte Ctd. Caja Quant. da caixa Ant. i kasse Aantal per doos Stck./Karton Ant. i esken Antal per låda	Pouch Qty Qté sachet Ctd. Bolsa Quant. da bolsa Ant. i hver pakning Aantal per zak Stck./Beutel Ant. i posen Antal per påse
PSSNB-CL	Clear/Transparent/Transparente/Klar/ Transparent/Gjennomsiktig/ Genomskinlig	60	1
PSSNB-AM	Amber/Orangé/Ámbar/Âmbar/Ravgul/ Oranje/Gyldenbrun/Bärnstensfärgad	60	1

Manufactured by:
 NeoMed Incorporated
 100 Londonderry Ct, Suite 112
 Woodstock, GA 30188 USA
 (770) 516-2225
www.neomedinc.com

Emergo Europe
 Molenaarstraat 15
 2513 BH, The Hague
 The Netherlands

ENGLISH

Indications for Use:

The device is indicated for use as a dispenser, a measuring device, an oral fluid transfer device and a collection and storage container.

Features

1. Compatible with most breast pumps, nipples and lids.
2. This device is not made with natural rubber latex, DEHP, or BPA.
3. All-in-one solution to simplify the collection, storage and delivery of expressed breast milk.
4. Eliminates the need to transfer breast milk between containers, and may lower the risk of contamination and infection.

Cautions

1. Do not wash and reuse NeoBottle™. It is intended for one time use only. Re-use of this device may increase risk of leakage or biological infection.
2. Do not heat NeoBottle™ in a microwave.
3. Do not heat NeoBottle™ in boiling water.

Collection

1. Wash hands thoroughly.
2. Remove NeoBottle™ from packaging.
3. Remove closed flip-top lid and retain for later use.
4. Attach NeoBottle™ to breast pump by screwing it onto the pump taking care not to over tighten.
5. Carefully express breast milk not exceeding maximum capacity of 80ml. Remove NeoBottle™ from pump and replace flip-top lid.

Storage and Feeding

1. Place the NeoBottle™ vertically upright on a flat surface.
2. Open the flip-top lid.
3. Fully depress the plunger of a NeoMed Oral Syringe and insert the syringe tip into the NeoBottle™ port.
4. Slowly pull the syringe plunger to evacuate the air from the NeoBottle™. As you pull the plunger, the floor of the NeoBottle™ will rise. Stop when milk enters the tip of the syringe.
5. Carefully remove the syringe to avoid spillage and close NeoBottle™ flip-top lid. It will click when securely closed.
6. Write the patient's name, date and time of milk collection in the space provided on the NeoBottle™.
7. If recording NeoBottle™ content volume, see note below.
NOTE: The gradient markings on the NeoBottle™ are designed to be read as a syringe using the leading edge of the gasket closest to the lid. To ensure accuracy when measuring, Steps 1-5 must be completed to remove all air (zero dead space) before volume measurement is made. Failure to do so may result in volume error.
8. Upright vertical storage is preferred. The filled NeoBottle™ can be stored in a refrigerator or freezer for later scheduled feedings.
9. The NeoBottle™ flip-top lid is not designed for direct feeding. To nipple feed, unscrew the flip-top lid and replace with a nipple and collar. The NeoBottle™ is compatible with a variety of standard nipples and collars. Do not re-use the NeoBottle™. After use, dispose of device per hospital protocols and/or local or national regulations.
- 10.

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